

# (MP)<sup>2</sup> TREATMENT INFORMED CONSENT BOOKLET

#### **INSTRUCTIONS**

This Informed Consent Booklet has been prepared by to help inform you about the potential benefits, associated risks, and alternatives of the Venus Versa<sup>™</sup> (MP) <sup>2</sup> treatment.

During your consultation and medical assessment, will have reviewed with you the potential benefits, associated risks, and alternatives of the Venus Versa<sup>™</sup> (MP)<sup>2</sup> treatment that are outlined in this booklet. They will have also provided you with answers to any and all questions you may have had about the procedure.

It is important that you read the information contained in this booklet again carefully and completely. Only when all of your questions and concerns about the procedures have been addressed should you then initial each page, indicating that you have read and fully understood all the items discussed in this booklet. When you reach the end of the booklet, please sign the consent for the procedure as proposed by

If you have any remaining questions or concerns about the potential benefits, associated risks, or alternatives of the Venus Versa<sup>™</sup> (MP)² treatment, do not initial any pages or sign the consent without speaking with

#### INTRODUCTION

Venus Versa<sup>™</sup> (MP)<sup>2</sup> treatments use a non-surgical radio frequency, pulsed electromagnetic device designed to tighten the skin, reduce circumference, and soften wrinkles. The device delivers radio frequency and pulsed electromagnetic energy targeted at the tissue, designed to damage the existing collagen, stimulating the body's healing response. Once the collagen has been damaged, the body begins to repair the skin by replacing the damaged tissue with new collagen. The radio frequency also stimulates the body to produce new fibroblasts, the "houses" that create collagen, thus increasing the amount of collagen in the tissue. This wound healing response creates a smoother appearance to the skin, plumps up fine lines and wrinkles, and treats acne. Pulsed electromagnetic fields increase the vascularity of the skin and create new pathways for the blood to flow. This increase in blood will provide more oxygen to the skin and bring more nutrients to the tissue. A fresher, more youthful glow is common after treatments.

# POTENTIAL BENEFITS OF THE VENUS VERSA™ (MP)<sup>2</sup> TREATMENT

The Venus Versa™ (MP)² treatment will aid in the improvement of the skin texture and appearance. While discussing the potential benefits this treatment with you, may have shown you a variety of before and after images. It is important to remind you that these images were used as an educational tool to allow you to visualize the general range skin improvements that may be achieved with your proposed treatment; the before and after images are not meant to be guarantees of actual or exact outcome.

# RISKS ASSOCIATED WITH THE VENUS VERSA™ (MP)2 TREATMENT

Every cosmetic procedure involves a very small degree of risk and, although exceedingly uncommon, it is important that you understand and accept the rare risks involved with the Venus Versa<sup>™</sup> (MP)<sup>2</sup> treatment. An individual's informed decision to undergo any cosmetic procedure is based upon a comparison of the risks against the potential benefits, alternatives and costs.

Although the vast majority of Venus Versa<sup>™</sup> (MP)<sup>2</sup> treatment patients never experience any of these complications, you should discuss each of them with to ensure you fully understand the alternatives, risks, potential complications, and average outcomes of the Venus Versa<sup>™</sup> (MP)<sup>2</sup> treatments.

•	Blisters - in rare cases, a blister may occur as a result of the treatment. In this instance,	
	will recommend	for the treatment of the

blister.

- Hyper- or Hypo- pigmentation in very rare cases a patient may experience changes in their skin colour which may or may not be permanent. In these cases,
   will recommend appropriate procedures which may address the changes in the appearance of the colour of the tissue.
- Swelling edema (or swelling of the skin) is common, and will resolve in a few days. Edema may occur as early as immediately post treatment and as late as a few days post treatment. It is advised to seek a consultation and follow up appointment with should you require medical attention or have concerns.

Client Initials:	Date:



There are many variable conditions, in addition to risks and potential complications listed above, that may influence the long-term result from the Venus Versa<sup>™</sup> (MP)<sup>2</sup> treatment. Even though risks and complications can occur infrequently, the risks cited in this booklet are particularly associated with the Venus Versa<sup>™</sup> (MP)<sup>2</sup> treatment. Other complications and risks can occur, but are even less common. Should complications occur, additional procedures or treatments may be necessary. The practice of medicine and surgery is not an exact science. Although good results are expected, there is no guarantee or warranty, expressed or implied, as to the results that may be obtained. Infrequently, it may be necessary to perform additional treatment to improve your results.

#### INFORMATION REGARDING PREGNANCY

Patients must not be pregnant, nor wish to become pregnant, for the duration of the treatment program. A reliable method of birth control is required, such as the birth control pill, diaphragm, condom, IUD, and/or abstinence. The effects on a pregnant patient or fetus undergoing this procedure have not been studied.

Please specify your method of birth control here:	

#### ALTERNATIVES TO THE VENUS VERSA™

#### **HEALTH INSURANCE**

Most health insurance companies exclude coverage for cosmetic procedures such as the Venus Versa™ (MP)² treatment. Health related complications that may arise from such treatment may not be covered by all insurance plans. Please carefully review your health insurance subscriber-information pamphlet, if you have a private insurance carrier.

# FINANCIAL RESPONSIBILITIES

Depending on whether the cost of treatment is covered by an insurance plan, you will be responsible for all necessary payments. Additional costs may occur should complications develop from treatment. There are no refunds once a treatment has been performed.

#### DISCLAIMER

Informed Consent Booklets are used to communicate information about the proposed treatment of a condition along with disclosure of risk and alternative treatment(s). The informed consent process attempts to define principles of risk disclosure that should generally meet the needs of most patients in most circumstances.

What has discussed with you and has been included in this booklet are the material risks both common and uncommon that feels a reasonable person would want to know, understand, and consider when making decisions to proceed with the proposed treatment.

However, Informed Consent Booklets should not be considered all-inclusive in defining other methods of care and risk encountered.

may provide you with additional or different information based on all the facts in your particular case and the state of medical knowledge.

Informed consent documents are not intended to define or serve as the standard of medical care. Standards of medical care are determined on the basis of all of the facts involved in an individual case, and are subject to change as scientific knowledge and technology advance, and as practice patterns evolve.

It is important that you read the above information contained on this and all preceding pages carefully and have all of your questions answered by before signing the consent on the last page.

Client Initials:	Date:	



# CONSENT FOR PROCEDURE AND/OR TREATMENT

#### I HAVE RECEIVED THE FOLLOWING INFORMATION/INFORMED CONSENT BOOKLET FOR: VENUS VERSA™ (MP)2 TREATMENT

ognize that due above. I then the exercise de all conditions sent to the ad	efore authorize the above physician and of his or her professional judgment neons that require treatment and are not known inistration of such topical anesthesia	and/or such assistants as may be selected to perform the nent unforeseen conditions may necessitate different procedures than d/or assistants or designees to perform such other procedures that sessary and desirable. The authority granted under this paragraph sha nown to my physician at the time the procedure is begun.		
e above. I then the exercise de all conditions sent to the ad	efore authorize the above physician and of his or her professional judgment neons that require treatment and are not known inistration of such topical anesthesia	d/or assistants or designees to perform such other procedures that sessary and desirable. The authority granted under this paragraph sha		
	sent to the administration of such topical anesthesia considered necessary or advisable. I understand that all forms of thesia involve risk and the possibility of complications, injury, and sometimes death.			
art of the requ	irements of the	, my chart may be subject to a peer review for quality contro		
nowledge that	no guarantee has been given by anyor	ne as to the results that may be obtained.		
I consent to the photographing or televising of the procedure(s) to be performed, including appropriate portions of my body, for medical, scientific or educational purposes, provided they do not reveal my identity. These photographs and videos may be used for medical meetings, advertising, or any promotional or public relations purposes.				
urposes of ad	vancing medical education, I consent to	o the admittance of observers to the treatment room.		
I understand that the signature of the witness (if a non-physician) on this document indicates only that the signing of my name has been observed and not that the witness has necessarily provided information regarding the procedure.				
IT HAS BEEN EXPLAINED TO ME BY MY PHYSICIAN AND/OR ASSISTANTS IN A WAY THAT I UNDERSTAND:				
i. ii. iii. iv.	THERE MAY BE ALTERNATIVE PRO THERE ARE RISKS TO THE PROCE	CEDURES OR METHODS OF TREATMENT		
I CON		TREATMENT AND THE ABOVE LISTED ITEMS (1-9). WITH THE EXPLANATION.		
Patient or	Person Authorized to Sign for Patient	Please Print Name Here		
	WITN	ESS:		
	sent to the phoal, scientific of edical meeting urposes of adderstand that the een observed AS BEEN EXP  i. ii. iii. iv.  I CON	cal, scientific or educational purposes, provided they edical meetings, advertising, or any promotional or purposes of advancing medical education, I consent to erstand that the signature of the witness (if a non-phosen observed and not that the witness has necessarial SEEN EXPLAINED TO ME BY MY PHYSICIAN AND I. THE ABOVE TREATMENT OR PROCIO. THERE MAY BE ALTERNATIVE PROCIO. THERE ARE RISKS TO THE PROCEIV. ANY QUESTIONS I MAY HAVE ASKED IV. ANY QUESTIONS I MAY HAVE ASKED I AM SATISFIED.		

\_ Date: \_

Client Initials: \_\_\_\_